

# reveal<sup>®</sup>

CLEAR ALIGNERS


 HENRY SCHEIN<sup>®</sup>  
DENTAL | ORTHODONTICS

 Rely on Us<sup>™</sup>

## How To Guide 3Shape Integration

[www.hsdorthodontics.eu](http://www.hsdorthodontics.eu) - [www.revealclearaligners.eu](http://www.revealclearaligners.eu) - 036 53 58 600 - [ortho@henryschein.nl](mailto:ortho@henryschein.nl)

**Step 1** Login to 3Shape Communicate, click on the “Connections” tab and search for your Aligner Lab via the lab’s main contact email or name.

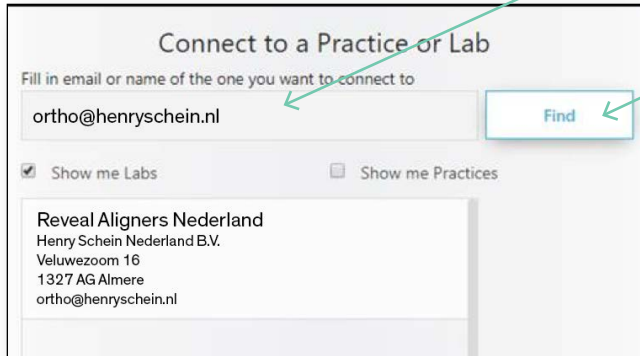


**Step 2** Under Settings, choose “Lab Connections” and select “Update”.


The Henry Schein Ortho lab account will appear as a lab on the practice’s TRIOS.

**Step 3** Logout.

**Step 4** Login to DDX, click on the drop-down arrow on the top right hand side of the page, to choose Store.

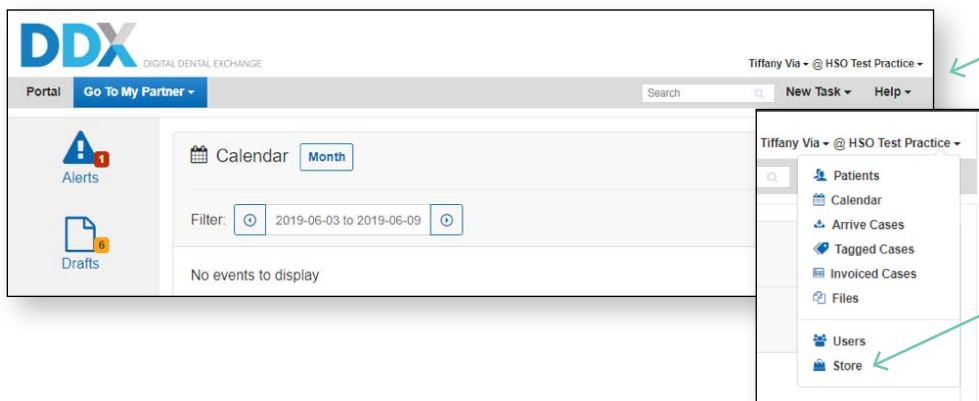

**Step 6** Scroll down and find TRIOS 3Shape Communicate and click on “Enable”.

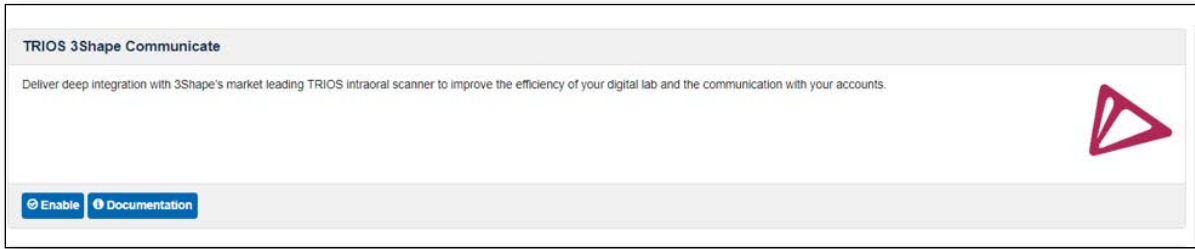

**Step 7** Click on “Configure your add-on”



**Step 8** Click on “Authorise” and type in the email and password of your Communicate account.



**Step 9** Now the two networks are connected! Before performing a TRIOS scan, select Reveal Clear Aligners as the receiver and DDX will create a Draft case.



**reveal**<sup>®</sup>  
CLEAR ALIGNERS

---

 **HENRY SCHEIN**<sup>®</sup>  
DENTAL | ORTHODONTICS

[www.hsdorthodontics.eu](http://www.hsdorthodontics.eu)

[www.revealclearaligners.eu](http://www.revealclearaligners.eu)

036 53 58 600

[ortho@henryschein.nl](mailto:ortho@henryschein.nl)